## INSTRUCTIONS FOR USE

# Dental LT Clear V2

<b>EN</b>	Instructions for Use	<b>FI</b>	Käyttöohjeet
<b>DE</b>	Gebrauchsanweisung	<b>PL</b>	Instrukcja używania
<b>FR</b>	Mode d'emploi	<b>CZ</b>	Návod k použití
<b>IT</b>	Istruzioni per l'uso	<b>TR</b>	Kullanım talimatları
<b>ES</b>	Instrucciones de uso	<b>EL</b>	Οδηγίες χρήσης
<b>PT</b>	Instruções de uso	<b>BG</b>	Инструкция за употреба
<b>NL</b>	Gebbruiksaanwijzing	<b>RU</b>	Указания по применению
<b>DK</b>	Anvisninger til brug	<b>JP</b>	使用説明書
<b>SV</b>	Bruksanvisning	<b>ZH</b>	使用说明

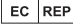
Prepared: 06/14/2022 PRNT-0028 Rev 01

## DENTAL LT CLEAR V2 RESIN

# Instructions for use

The following instructions for use are for Formlabs photopolymer Dental LT Clear V2 Resin. Basic information about safety and environmental concerns are also included. For more detailed safety and environment information please refer to the Safety Data Sheet, available at [dental.formlabs.com](http://dental.formlabs.com). For further information regarding the use of this material, please contact Formlabs.

Prepared: 06/14/2022 PRNT-0028 Rev 01

	: Keep away from sunlight		: Use-by date
	: Consult Instructions for Use		: European Conformity
	: Batch Code		: Catalog Number
	: Manufacturer		: Temperature Limit
	: Authorized representative in the European Community		: Medical Device
	: Environmental hazard		: Warning: Read Safety Information
	: Unique device identifier		

 Formlabs Ohio Inc.  
27800 Lemoyne Rd,  
Millbury, OH 43447 USA  
+1 617 855 0762

EU Importer  
Formlabs GmbH  
Funkhaus Berlin  
Nalepastr. 18  
12459 Berlin, Germany  
+49 1573 5993322

## 1. Introduction And Indications For Use

### INTRODUCTION

Dental LT Clear V2 Resin is a light-curable polymer-based resin designed for the fabrication of biocompatible, long-term use, removable dental and orthodontic appliances by additive manufacturing.

### INTENDED PURPOSE

It is intended for 3D printing dental and orthodontic appliances such as occlusal splints or night guards.

## INDICATIONS FOR USE

Dental Clear LT V2 Resin when utilized to print dental or orthodontic appliances such as occlusal splints or night guards is indicated to treat patients diagnosed with Temporomandibular Joint Disorders (TMD) and/or Bruxism, respectively.

## CONTRAINDICATIONS

Dental LT Clear V2 Resin should not be used in case of any known allergy or hypersensitivity history to photopolymer resin materials. In case of an allergic reaction, stop use immediately, and consult a physician.

## INTENDED PATIENT POPULATION

All patients and populations in which a clinician has prescribed a treatment associated with the intended purpose.

## INTENDED USERS

Dental and orthodontic professionals who fabricate dental and orthodontic appliances.

## CLINICAL BENEFIT

The clinical benefit of Dental LT Clear V2 Resin is the ability to print dental or orthodontic appliances such as occlusal splints or night guards.

## 2. Safety

### RESIN ASSOCIATED WARNINGS

Dental LT Clear V2 Resin (uncured) contains polymerizable monomers which may cause skin irritation (allergic contact dermatitis) or other allergic reactions in susceptible persons. If resin contacts skin, wash thoroughly with soap and water. If skin sensitization occurs, discontinue use. If dermatitis or other symptoms persist, seek medical assistance.

1. **Eye contact:** High vapor concentration may cause irritation.
2. **Skin contact:** May cause sensitization by skin contact. Irritating to skin. Repeated and/or prolonged contact may cause dermatitis.
3. **Inhalation:** Irritating to the respiratory system. Prolonged or repeated exposure may cause: headache, drowsiness, nausea, weakness (severity of effects depends on the extent of exposure).
4. **Ingestion:** Low oral toxicity, but ingestion may cause irritation of the gastrointestinal tract.
5. **Protection:** Protective glasses and nitrile gloves should be worn while handling Dental LT Clear V2 Resin. Detailed information about the handling of Dental LT Clear V2 Resin can be found in the Safety Data Sheets at [dental.formlabs.com](http://dental.formlabs.com).

### RESIN ASSOCIATED PRECAUTIONS

1. When washing the printed part with solvent, it should be in a properly ventilated environment with proper protective masks and gloves.
2. Expired or unused Dental LT Clear V2 Resin shall be disposed of in accordance with local regulations.

3. IPA shall be disposed of in accordance with local regulations.
4. Disposal recommendations in Section 5. G.

### INFORMATION FOR PATIENT SAFETY

1. If signs of extreme wear out, deterioration and/or perforations appear, the appliances should be replaced to avoid material failure and potential bodily injury.
2. The patient should inform the clinician of any known allergy or hypersensitivity history to photopolymer resin materials.
3. In case of an allergic reaction, stop use immediately, and consult a medical physician.

### 3. Hardware And Printing Parameters

- a. **Hardware:** Formlabs SLA 3D Printer (Laser Wavelength: 405 nm)
- b. **Software:** Formlabs PreForm
  - STL file import
  - Manual / Automatic rotation and placement
  - Manual / Automatic generation of supports
- c. **Printing Parameters**
  - **Orientation:** Intaglio surface facing away from the build platform at a 0 - 40° tilted angle
  - Minimum occlusal surface thickness of 2 mm
  - Minimum wall thickness of 1 mm
- d. **Recommended Post-Processing Equipment:**
  - Formlabs validated Wash Unit
  - Isopropyl alcohol (IPA) ≥ 99%
  - Formlabs validated Post Cure Unit

### 4. Performance Characteristics

Mechanical Properties	Post Cured Results <sup>1</sup>	Method
Ultimate Tensile Strength	≥ 50 MPa	ASTM D638
Flexural Modulus	≥ 2000 MPa	ASTM D790
Elongation	≥ 10%	ASTM D638

Dental LT Clear V2 Resin has been evaluated in accordance with EN ISO 10993-1:2020, Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process, and ISO 7405:2018, Dentistry - Evaluation of biocompatibility of medical devices used in dentistry, and passed the requirements for the following biocompatibility risks:

ISO Standard	Description <sup>2</sup>
ISO 10993-5	Not cytotoxic
ISO 10993-10	Not an irritant
ISO 10993-10	Not a sensitizer
ISO 10993-3	Not mutagenic
ISO 10993-17, ISO 10993-18	Not toxic (subacute / subchronic)

<sup>1</sup> Material properties may vary based on part geometry, print orientation, print settings, temperature, and disinfection or sterilization methods used.

<sup>2</sup> Dental LT Clear V2 Resin was tested at NAMSA World Headquarters, OH, USA.

## 5. Specific Manufacturing Considerations

### NOTIFICATION

The device specifications have been validated using the printer process parameters indicated below.

### REQUIREMENTS

Use dedicated accessories for Dental LT Clear V2 Resin. For biocompatibility compliance, Dental LT Clear V2 Resin requires a dedicated resin tank, build platform, Formlabs validated wash unit, and finishing kit, which should not be mixed with any other resins.

#### A. PRINTING

1. **Shake cartridge:** Prior to use, shake the cartridge well. Color deviations and print failures may occur if the cartridge is shaken insufficiently.
2. **Set up:** Insert resin cartridge into a compatible Formlabs 3D printer.
3. **Printing:**
  - a. Prepare a print job using PreForm software. Import desired part STL file. Orient and generate supports. For recommendations on print orientation and support placement, visit [dental.formlabs.com](https://dental.formlabs.com).
  - b. Send the print job to the printer. Begin print by selecting a print job from the print menu. Follow any prompts or dialogs shown on the printer screen. Printer will automatically complete the print.
4. **Cartridge Reuse:**
  - a. After the print is completed, store unused resin cartridges as instructed in Section F below.
  - b. For a new print job, make sure to shake the cartridge well and follow all instructions as recommended.

#### B. POST-PROCESSING

**Warning:** During production, inspect the printed appliance for cracks, rough edges, or any other artifacts that may affect finished device performance or patient comfort and safety. Part inspection is recommended after every fabrication step, for example but not limited to, after printing, support removal, cleaning, and/or sterilization when applicable.

1. **Part removal:**
  - a. Remove the build platform from the printer.
  - b. Printed parts can be removed from the build platform before or after cleaning. To remove, wedge the part removal tool under the printed part raft, and rotate the tool. For detailed techniques visit [support.formlabs.com](https://support.formlabs.com).
2. **Rinsing:** Required - Place the printed parts in a Formlabs validated wash unit filled with Isopropyl Alcohol (IPA,  $\geq 99\%$ ) and wash for 15 minutes. Remove parts from the wash unit and soak in fresh Isopropyl Alcohol (IPA,  $\geq 99\%$ ) for 5 minutes.
3. **Drying:**
  - a. Remove parts from IPA and leave to air dry at room temperature for at least 30 minutes.  
**Note:** Dry times can vary depending on design of parts and ambient conditions.
  - b. Inspect printed parts to ensure that parts are clean and dry. No residual alcohol, excess liquid resin or residue particles must remain on the surface before proceeding to subsequent steps.
  - c. If residual alcohol is still present, dry parts longer. If resin residue is still visible, rewash parts until clean and dry.
4. **Post cure:** Required - Place the dried printed parts in a Formlabs validated post curing unit and post cure at 60°C for 60 minutes. Allow the post curing unit to cool down to room temperature between cure cycles.

#### 5. **Support removal:**

- a. Remove supports using a cutting disk and handpiece, or with other part removal tools.
- b. Inspect the parts for any cracks. Discard if any damage or cracks are detected.

### **C. POLISHING**

1. If any rough marks are left on the printed splint surface after support removal, smooth down the support surfaces with a bur and handpiece to improve patient comfort.
2. Polish the printed splints using typical dental polishing methods prior to patient use.

### **D. CLEANING**

1. Fully post processed parts can be cleaned using a dedicated soft toothbrush with neutral soap and room temperature water.
2. Do not use toothpaste or any other abrasive cleaning products on parts printed with Dental LT Clear V2 Resin. Such cleaning products may affect surface finish and patient comfort.
3. After cleaning, always inspect parts for any cracks. Discard if any damage or cracks are detected.

### **E. DISINFECTION**

1. The appliances may be cleaned and disinfected according to facility protocols. Tested method of disinfection includes: soaking the finished appliance in fresh 70% IPA for 5 minutes.  
**Note:** Do not leave the part in the alcohol solution for longer than 5 minutes.
2. After disinfection, inspect the part for cracks to ensure the integrity of the impression tray.

### **F. STORAGE**

1. When not in use, place printed parts in closed, opaque or amber containers.
2. Store in a cool, dry place out of direct sunlight. Excess light exposure overtime may affect the color of printed parts.
3. Store the cartridges at 10°C - 25°C (50°F - 77°F).
4. Do not exceed 25°C (77°F) when in storage.
5. Keep away from ignition sources.

### **G. DISPOSAL**

1. Any cured resin is non-hazardous and may be disposed of as regular waste.
2. Follow clinic or facility protocols for waste that may be considered biohazardous.
3. Liquid resin should be disposed of in accordance with government regulations (community, regional, national).
  - a. Contact a licensed professional waste disposal service to dispose of liquid resin.
  - b. Do not allow waste to enter storm or sewer drainage systems.
  - c. Avoid release into the environment.
  - d. Contaminated packaging: Dispose of cartridge or any unused product per local waste management guidelines.

### **H. ADVERSE EVENTS**

To report any serious injury (temporary or permanent serious deterioration of health, a serious public health threat, or death), contact Formlabs Inc. (+1 617 855 0762) or your local Competent Authority <sup>3</sup>.

---

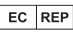
3 Local EU Competent Authorities listed here: [https://ec.europa.eu/health/sites/health/files/md\\_sector/docs/md\\_vigilance\\_contact\\_points.pdf](https://ec.europa.eu/health/sites/health/files/md_sector/docs/md_vigilance_contact_points.pdf) md\_vigilance\_contact\_points.pdf

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Formlabs Ohio Inc.  
27800 Lemoyne Rd,  
Millbury, OH 43447, USA  
+1 617 855 0762

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